



Web Based

Stock Management and Temperature Monitoring Software

A photograph of a smiling man in a light blue shirt sitting at a desk, looking at a laptop. Overlaid on the right side of the image are four browser icons (Google Chrome, Internet Explorer, Firefox, and Safari) and the Sensorium logo, all hanging from thin black lines as if they were weights on a string. The background of the photo is a blurred office setting.


Temperature analysis can be made in any place with an **internet connection**.
System includes web based **Datamatrix**-supported stock tracking.

User Guide




www.sensorium.com.tr

Main Screen

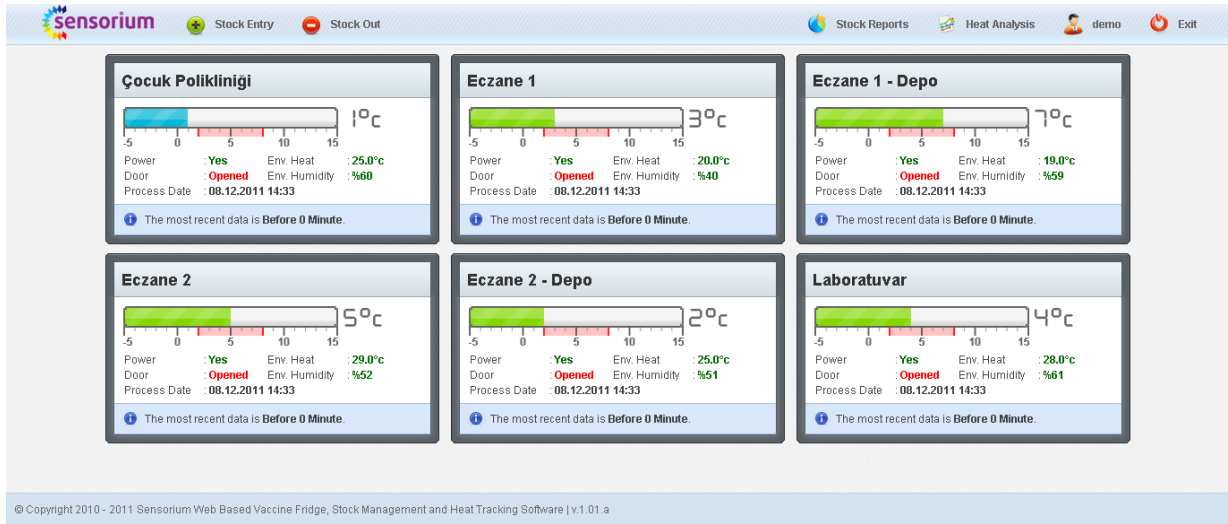

In every 5 minutes, you can view the instant temperature and humidity values.

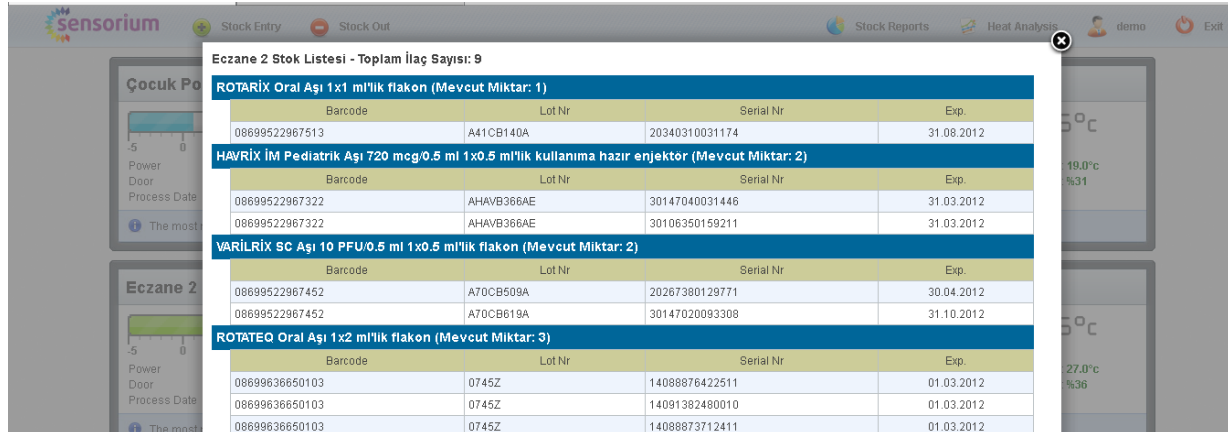

Eczane 2 Stok Listesi - Toplam İlaç Sayısı: 9

Barcode	Lot Nr	Serial Nr	Exp.
ROTARIX Oral Aşı 1x1 ml'lik flakon (Mevcut Miktar: 1)			
08699522967513	A41CB140A	20340310031174	31.08.2012
HAVRIX İM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör (Mevcut Miktar: 2)			
08699522967322	AHAVB366AE	30147040031446	31.03.2012
08699522967322	AHAVB366AE	30106350159211	31.03.2012
VARİLRIX SC Aşı 10 PFU/0.5 ml 1x0.5 ml'lik flakon (Mevcut Miktar: 2)			
08699522967452	A70CB509A	20267380129771	30.04.2012
08699522967452	A70CB619A	30147020093308	31.10.2012
ROTATEQ Oral Aşı 1x2 ml'lik flakon (Mevcut Miktar: 3)			
08699636650103	0745Z	14088876422511	01.03.2012
08699636650103	0745Z	14091382480010	01.03.2012
08699636650103	0745Z	14088873712411	01.03.2012

By clicking on the fridge name, you can view the instant stock status.

Stock Entry

sensorium Stock Entry Stock Out Stock Reports Heat Analysis demo Exit

Stock Entry

Please Select Fridge

Please Select Fridge

Eczane 2

Eczane 1

Eczane 1 - Depo

Eczane 2 - Depo

Çocuk Polikliniği

Laboratuvar

Click here for stock entry with datamatrix reader.

Select the fridge to stock entry.

sensorium Stock Entry Stock Out Stock Reports Heat Analysis demo Exit

Stock Entry

Eczane 2

Use the datamatrix reader to make the stock entry.

Stock Out

sensorium Stock Entry Stock Out Stock Reports Heat Analysis demo Exit

Stock Out

Please Select Fridge

Please Select Fridge

Eczane 2

Eczane 1

Eczane 1 - Depo

Eczane 2 - Depo

Çocuk Polikliniği

Laboratuvar

Click here for stock out with datamatrix

Select the fridge to stock out.

sensorium Stock Entry Stock Out Stock Reports Heat Analysis demo Exit

Stock Out

Eczane 2

By using the datamatrix reader, make the stock out.

Transaction Reports

sensorium Stock Entry Stock Out Stock Reports Heat Analysis demo Exit

Stock Report

Start Date

End Date

Fridge: Eczane 2, Eczane 1, Eczane 1 - Depo, Eczane 2 - Depo

Report Type: Transaction Report

Submit

Select the stock report type.

Select the timeline.

Select the fridge that you want to get report.

Click here to see transaction and stock reports.

sensorium Stock Entry Stock Out Stock Reports Heat Analysis demo Exit

Stock Report

Start Date

End Date

Fridge: Eczane 1 - Depo, Eczane 2 - Depo, Çocuk Polikliniği, Laboratuvar

Report Type: Transaction Report

Submit

Click here to see your report.

sensorium Stock Entry Stock Out Stock Reports Heat Analysis Admin Demo Exit







Copy CSV Excel PDF Print

Show 25 entries

Search:

Fridge	Vaccine Name	Barcode	Lot Nr	Serial Nr	Exp.	User	Date	Operation
Eczane Depo 1-65	ROTARİX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780001159	121031	Erkan Tokyürek	05.10.2011 08:27	Output
Eczane Depo 1-65	ROTARİX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780001567	121031	Erkan Tokyürek	05.10.2011 08:26	Output
Eczane Depo 1-65	ROTARİX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780001607	121031	Erkan Tokyürek	05.10.2011 08:26	Output
Eczane Depo 1-65	ROTARİX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780001199	121031	Erkan Tokyürek	05.10.2011 08:26	Output
Eczane Depo 1-65	ROTARİX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780001239	121031	Erkan Tokyürek	05.10.2011 08:26	Output
Eczane Depo 1-65	ROTARİX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780001655	121031	Erkan Tokyürek	05.10.2011 08:26	Output
Eczane Depo 1-65	ROTARİX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780000983	121031	Erkan Tokyürek	05.10.2011 08:26	Output

Stock Reports

sensorium      


Stock Report







Start Date





End Date

Fridge
Eczane 1
Eczane 1 - Depo
Eczane 2 - Depo

Report Type

[Click here to see your report.](#) 







sensorium      


Copy    





Show entries

Search:

Fridge	Vaccine Name	Barcode	Lot Nr	Serial Nr	Exp.
Eczane Depo 1-65	ROTARIX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780009303	31.10.2012
Eczane Depo 1-65	ROTARIX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780009335	31.10.2012
Eczane Depo 1-65	ROTARIX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780009343	31.10.2012
Eczane Depo 1-65	ROTARIX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780009311	31.10.2012
Eczane Depo 1-65	ROTARIX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780009351	31.10.2012
Eczane Depo 1-65	ROTARIX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780000543	31.10.2012
Eczane Depo 1-65	ROTARIX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780000535	31.10.2012
Eczane Depo 1-65	ROTARIX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780000743	31.10.2012
Eczane Depo 1-65	ROTARIX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780000783	31.10.2012
Eczane Depo 1-65	ROTARIX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780000815	31.10.2012
Eczane Depo 1-65	ROTARIX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780000823	31.10.2012

sensorium      

[Click here to print the report.](#) 

Copy    

Show entries

Search:

Fridge	Vaccine Name	Barcode	Lot Nr	Serial Nr	Exp.	User	Date	Operation
Eczane Depo 1-65	ROTARIX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780001159	121031	Erkan Tokyürek	05.10.2011 08:27	Output
Eczane Depo 1-65	ROTARIX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780001567	121031	Erkan Tokyürek	05.10.2011 08:26	Output
Eczane Depo 1-65	ROTARIX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780001607	121031	Erkan Tokyürek	05.10.2011 08:26	Output
Eczane Depo 1-65	ROTARIX Oral Aşı 1x1 ml'lik flakon	08699522967513	A41CB137A	20363780001199	121031	Erkan Tokyürek	05.10.2011 08:26	Output

Fridge	Vaccine Name	Barcode	Lot Nr	Serial Nr	Exp.	User	Date	Operation
Eczane Depo 2-68	ROTATEQ Oral Aşı 1x2 ml'lik flakon	08699636650103	0745Z	14088873702211	120301	Erkan Tokyürek	29.10.2011 21:41	Input
Eczane Depo 2-68	ROTATEQ Oral Aşı 1x2 ml'lik flakon	08699636650103	0745Z	14088872204911	120301	Erkan Tokyürek	29.10.2011 21:40	Output
Eczane Depo 2-68	ROTATEQ Oral Aşı 1x2 ml'lik flakon	08699636650103	0745Z	14088872338211	120301	Erkan Tokyürek	29.10.2011 21:40	Output
Eczane Depo 2-68	ROTATEQ Oral Aşı 1x2 ml'lik flakon	08699636650103	0745Z	14088872351511	120301	Erkan Tokyürek	29.10.2011 21:40	Output
Eczane Depo 2-68	ROTATEQ Oral Aşı 1x2 ml'lik flakon	08699636650103	0745Z	14088872351511	120301	Erkan Tokyürek	29.10.2011 21:40	Output
Eczane Depo 2-68	ROTATEQ Oral Aşı 1x2 ml'lik flakon	08699636650103	0745Z	14088872351511	120301	Erkan Tokyürek	29.10.2011 21:39	Output
Eczane Depo 2-68	ROTATEQ Oral Aşı 1x2 ml'lik flakon	08699636650103	0745Z	14088872351511	120301	Erkan Tokyürek	29.10.2011 21:39	Output
Eczane Depo 2-68	ROTATEQ Oral Aşı 1x2 ml'lik flakon	08699636650103	0745Z	14088872351511	120301	Erkan Tokyürek	29.10.2011 21:39	Output
Eczane Depo 2-68	HAVRIX İM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699522967322	AHAVB366AE	30197390152746	120331	Erkan Tokyürek	30.09.2011 08:32	Output
Eczane Depo 2-68	HAVRIX İM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699522967322	AHAVB366AE	30197390152746	120331	Erkan Tokyürek	30.09.2011 08:32	Output

Print Preview

Please use your browser's PRINT FUNCTION to printing. After printing you can return main screen with ESC(escape) button.

Stock Reports

sensorium Stock Entry Stock Out Stock Reports Heat Analysis Admin Demo Exit

Click here to save report in pdf format.

Show 25 entries Search:

Fridge	Vaccine Name	Barcode	Lot Nr	Serial Nr	Exp.
Eczane Depo 2- 68	GARDASIL IM Hazir Enjektör 0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	08699636950104	NM31320	14078234779310	27.03.2012
Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	08699522967322	AHAVB366AE	30197390041320	31.03.2012
Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	08699522967322	AHAVB366AE	30197390041306	31.03.2012
Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	08699522967322	AHAVB366AE	30197390041250	31.03.2012
Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	08699522967322	AHAVB366AE	30197390041236	31.03.2012

Sensorium.pdf - Adobe Reader

File Edit View Window Help

Tools Comment

Sensorium

Stock Report

Fridge	Vaccine Name	Barcode	Lot Nr	Serial Nr	Exp.
Eczane Depo 2- 68	GARDASIL IM Hazir Enjektör 0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	08699636950104	NM31320	14078234779310	27.03.2012
Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	08699522967322	AHAVB366AE	30197390041320	31.03.2012
Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	08699522967322	AHAVB366AE	30197390041306	31.03.2012
Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	08699522967322	AHAVB366AE	30197390041250	31.03.2012
Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	08699522967322	AHAVB366AE	30197390041236	31.03.2012

Converting the report to pdf.

sensorium Stock Entry Stock Out Stock Reports Heat Analysis Admin Demo Exit

Click here to save the report xls format.

Show 25 entries Search:

Fridge	Vaccine Name	Barcode	Lot Nr	Serial Nr	Exp.
Eczane Depo 2- 68	GARDASIL IM Hazir Enjektör 0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	08699636950104	NM31320	14078234779310	27.03.2012
Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	08699522967322	AHAVB366AE	30197390041320	31.03.2012
Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	08699522967322	AHAVB366AE	30197390041306	31.03.2012
Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	08699522967322	AHAVB366AE	30197390041250	31.03.2012
Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	08699522967322	AHAVB366AE	30197390041236	31.03.2012

Sensorium - Microsoft Excel

Fridge	Vaccine Name	Barcode	Lot Nr	Serial Nr	Exp.
1 Eczane Depo 2- 68	GARDASIL IM Hazir Enjektör 0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	8699636950104	NM31320	14078234779310	27.03.2012
2 Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	8699522967322	AHAVB366AE	30197390041320	31.03.2012
3 Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	8699522967322	AHAVB366AE	30197390041306	31.03.2012
4 Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	8699522967322	AHAVB366AE	30197390041250	31.03.2012
5 Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	8699522967322	AHAVB366AE	30197390041236	31.03.2012
6 Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanima hazır enjektör	8699522967322	AHAVB366AE	30197390041180	31.03.2012

Converting the report to Excel.

Stock Reports

sensorium Stock Entry Stock Out Stock Reports Heat Analysis Admin Demo Exit

Click here to copy the report, then paste it to an excel document.

Show 25 entries

Search:

Fridge	Vaccine Name	Barcode	Lot Nr	Serial Nr	Exp.
Eczane Depo 2- 68	GARDASIL IM Hazır Enjektör 0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699636950104	NM31320	14078234779310	27.03.2012
Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699522967322	AHAVB366AE	30197390041320	31.03.2012
Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699522967322	AHAVB366AE	30197390041306	31.03.2012
Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699522967322	AHAVB366AE	30197390041250	31.03.2012
Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699522967322	AHAVB366AE	30197390041236	31.03.2012


Copy CSV Excel PDF Print

Kitap1 - Microsoft Excel

	A	B	C	D	E	F	G	H	I	J	K
1	Fridge	Vaccine Name	Barcode	Lot Nr	Serial Nr	Exp.					
2	Eczane Depo 2- 68	GARDASIL IM Hazır Enjektör 0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699636950104	NM31320	14078234779310	27.03.2012					
3	Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699522967322	AHAVB366AE	30197390041320	31.03.2012					
4	Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699522967322	AHAVB366AE	30197390041306	31.03.2012					
5	Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699522967322	AHAVB366AE	30197390041250	31.03.2012					
6	Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699522967322	AHAVB366AE	30197390041236	31.03.2012					
7	Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699522967322	AHAVB366AE	30197390041180	31.03.2012					
8	Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699522967322	AHAVB366AE	30197390041166	31.03.2012					
9	Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699522967322	AHAVB366AE	30197390041110	31.03.2012					
10	Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699522967322	AHAVB366AE	30197390041096	31.03.2012					
11	Eczane Depo 2- 68	HAVRIX IM Pediatrik Aşı 720 mcg/0.5 ml 1x0.5 ml'lik kullanıma hazır enjektör	08699522967322	AHAVB366AE	30197390041040	31.03.2012					

Copying the report to excel.

Heat Analysis

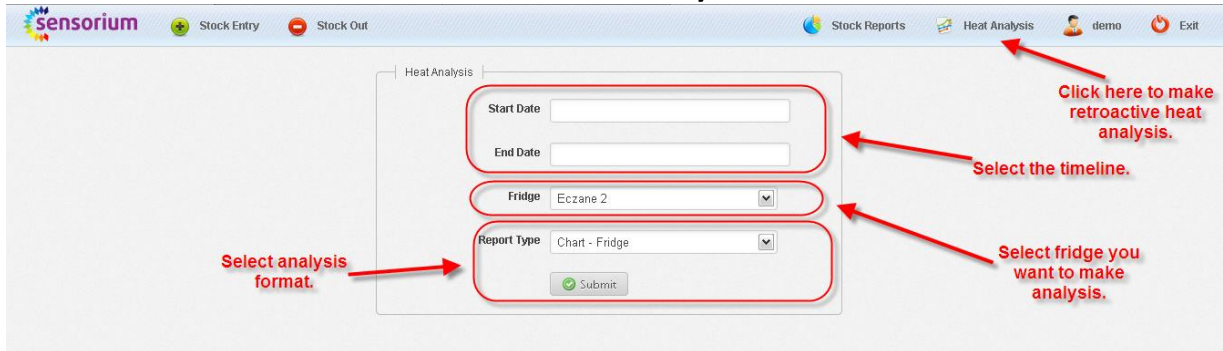


The form is titled "Heat Analysis" and contains the following fields:

- Start Date**: Text input field.
- End Date**: Text input field.
- Fridge**: Dropdown menu with "Eczane 2" selected.
- Report Type**: Dropdown menu with "Chart - Fridge" selected.
- Submit**: Button with a green checkmark icon.

Annotations with red arrows point to the form fields:

- "Click here to make retroactive heat analysis." points to the "Heat Analysis" link in the top navigation bar.
- "Select the timeline." points to the "Start Date" and "End Date" fields.
- "Select fridge you want to make analysis." points to the "Fridge" dropdown menu.
- "Select analysis format." points to the "Report Type" dropdown menu.

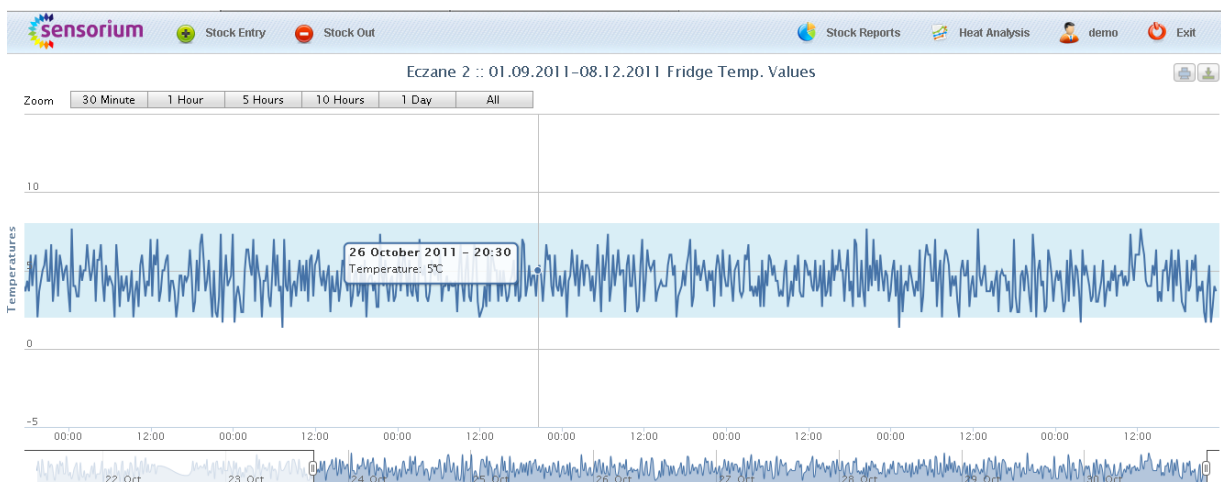

Heat analysis charts.




The table displays heat analysis data for "Eczane 2". It includes columns for Fridge, Temperature, Env. Heat, Env. Humidity, Power, Door, and Date. The table is sorted by Date in descending order.

Fridge	Temperature	Env. Heat	Env. Humidity	Power	Door	Date
Eczane 2	4.00	26.00	56.00	Yes	Opened	21.09.2011 11:14
Eczane 2	4.00	27.00	54.00	Yes	Opened	21.09.2011 11:15
Eczane 2	6.00	10.00	64.00	Yes	Opened	21.09.2011 11:20
Eczane 2	8.00	19.00	34.00	Yes	Opened	21.09.2011 11:25
Eczane 2	2.00	19.00	52.00	Yes	Opened	21.09.2011 11:30
Eczane 2	4.00	33.00	36.00	Yes	Opened	21.09.2011 11:35
Eczane 2	6.00	32.00	63.00	Yes	Opened	21.09.2011 11:40

Heat analysis lists.

Fridge Definations

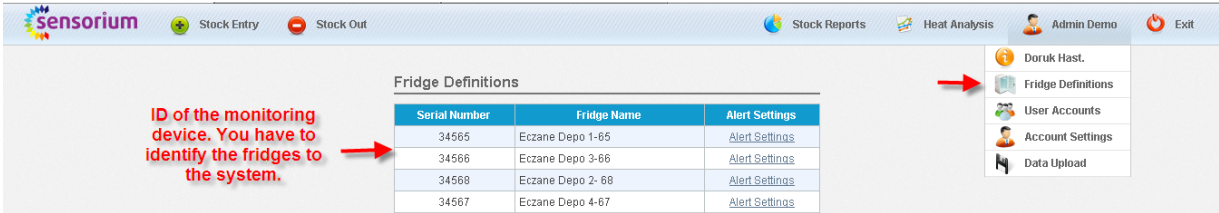

sensorium Stock Entry Stock Out Stock Reports Heat Analysis Admin Demo Exit

Fridge Definitions

Serial Number	Fridge Name	Alert Settings
34565	Eczane Depo 1-65	Alert Settings
34566	Eczane Depo 3-66	Alert Settings
34568	Eczane Depo 2- 68	Alert Settings
34567	Eczane Depo 4-67	Alert Settings

[Doruk Hast.](#)
[Fridge Definitions](#)
[User Accounts](#)
[Account Settings](#)
[Data Upload](#)

ID of the monitoring device. You have to identify the fridges to the system.

User Accounts



sensorium Stock Entry Stock Out Stock Reports Heat Analysis Admin Demo Exit

Copy CSV Print


Show 25 entries

Name	Mobile	E-mail	Language	Group
Admin Demo	0 (533) 445 92 66	fehmi.buge@gmail.com	en	Administrator
Burak CEBEL	0 (544) 642 04 81	burakcebel@trex.com.tr	tr	User

[Doruk Hast.](#)
[Fridge Definitions](#)
[User Accounts](#)
[Account Settings](#)
[Data Upload](#)

Define the users that are allowed to enter the system.

User Accounts and Authorities

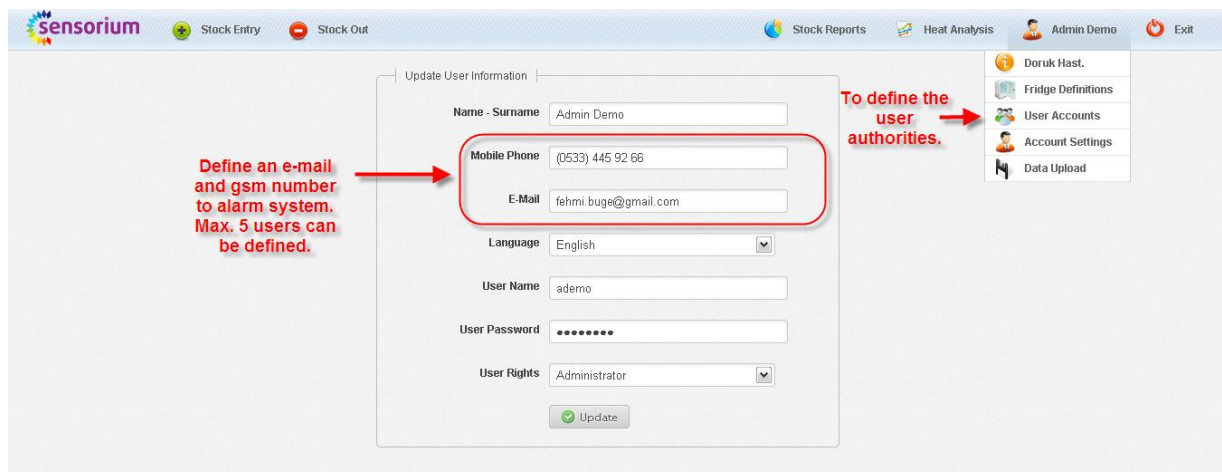

sensorium Stock Entry Stock Out Stock Reports Heat Analysis Admin Demo Exit

Update User Information

Name - Surname Admin Demo

Mobile Phone (0533) 445 92 66

E-Mail fehmi.buge@gmail.com

Language English

User Name ademo

User Password

User Rights Administrator

[Update](#)

[Doruk Hast.](#)
[Fridge Definitions](#)
[User Accounts](#)
[Account Settings](#)
[Data Upload](#)

Define an e-mail and gsm number to alarm system. Max. 5 users can be defined.

To define the user authorities.

